

Patient Application Instructions


BEFORE USING THE DEVICE, PLEASE READ THE **INSTRUCTIONS FOR USE COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

APPLICATION INSTRUCTIONS

The product is designed to be worn during normal everyday activities.




1. Open the belt.

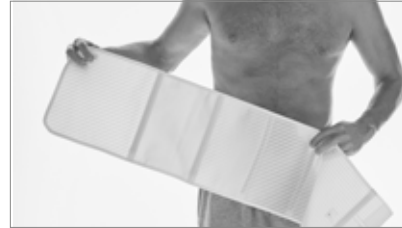

2. Position the abdominal support around the waist. The non-ribbed section of elastic material can be placed over an area that will require access, for example for a probe or catheter insertion. An opening can be cut through the non-ribbed section of elastic material (without causing fraying) to allow access. The non-ribbed section can be positioned on the left or right side of the body by turning it upside down. Note: The sections of ribbed elastic material cannot be cut.



3. While maintaining the abdominal support in position, pull on the hook-and-loop closure tab and fix it onto the abdominal support where best suited for closure.

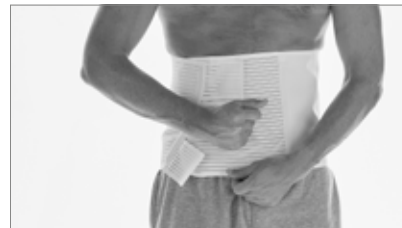

4. Adjust the hook-and-loop closure tab until desired support is obtained.



5. The 3 front stays can be removed through the slots.



6. The level of compression can be readjusted as needed during the day.

REMOVABLE ELASTIC BAND

For additional support of the abdomen, use the removable elastic band.

Position the elastic band on the central part of the abdominal support and fix it with the hook-and-loop fasteners.

Pull on both edges of the elastic band, until desired level of comfort and support, then fix them onto the abdominal support using the hook-and-loop fasteners.



ACCESSORIES

The DonJoy Abdostrap 2.0 has been designed to be used with the following accessories that are optional:

- Umbilical hernia pad.
- Abdominal hernia pad.

Both pads are made with antibacterial, antimicrobial fabric and hook-and-loop fasteners for easy positioning.

INTENDED USE / INDICATIONS


The DonJoy Abdostrap 2.0 is designed to provide comfort and support to the abdomen. It may be suitable for strains or muscle weakness of the abdominal area and for post operative abdominal surgeries. Provides abdominal support following abdominal and gynecological surgery, such as: Laparoendoscopic Single-Site Surgery, cesarean delivery, provides support for muscle strains or weakness of the of the abdominal area.

WARNINGS AND PRECAUTIONS

- We recommend that the first application be made under supervision of a healthcare professional.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult a healthcare professional immediately.

ORDERING INFORMATION

PART NUMBER			WAIST CIRCUMFERENCE	SIZE
HEIGHT 18CM	HEIGHT 25CM	HEIGHT 32CM		
82-0076-2	82-0077-2	82-0078-2	68 - 82 cm	S
82-0076-3	82-0077-3	82-0078-3	82 - 98 cm	M
82-0076-4	82-0077-4	82-0078-4	98 - 116 cm	L
82-0076-5	82-0077-5	82-0078-5	116 - 136 cm	XL
82-0076-6	82-0077-6	82-0078-6	136 - 156 cm	XXL

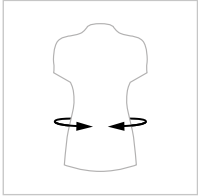

CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max 40°C) with a mild detergent.

INTENDED FOR SINGLE PATIENT MULTIPLE USE

NOT MADE WITH NATURAL RUBBER LATEX

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.