

PROCARE®

Healing & Recovery



Off-Loading Diabetic Shoe



► OFF-LOADING DIABETIC SHOE

One of the major adverse outcomes of diabetes is neuropathy and foot ulcers. Up to 85% of all amputations from diabetes begin with an ulcer and every year, approximately 4 million more people develop a diabetic foot ulcer. (International Diabetes Federation: 2008).

The new Procure® Diabetic Shoe can be used in the treatment of diabetic ulcers. The shoe includes the IMPAX insole which is constructed of pre-cut layered Plastazote foam. The bottom two layers of foam can be easily removed around the affected areas which help to relieve pressure from specific areas and so promote wound healing. Another key benefit in the design is that the top layer of foam stays in contact with the plantar surface of the foot, not allowing “hot spots” and further complications.

The shoe can be easily taken off and put on as needed to allow access to the affected area. It is lightweight and the rocker sole promotes normal ambulation. The IMPAX sole can be removed and replaced to ensure a hygienic environment.

► INDICATIONS

- Treatment of ulcerative or pre-ulcerative conditions of the foot
- Alternative treatment when Diabetic Walker or Total Contact Casting is not an option

► FEATURES AND BENEFITS

- Tri-laminate IMPAX grid insole designed to redistribute load away from ulcers with easy to remove IMPAX foam blocks
- IMPAX grid allows for a continuous plantar surface
- Lightweight breathable material enhances patient's comfort and promotes compliance
- Squared toe box and wide foot bed to leave ample room for wound dressings
- Removable reinforced toe cover that protects the toes from external aggression, whilst allowing easy access to the treatment area
- Rocker bottom helps to redistribute the weight of the patient through the plantar surface and off-load the wound area
- Plastazote foam is simple to clean and maintain
- Universal left and right minimises inventory




► ORDERING INFORMATION

Part Number	Description	Male Shoe Size EU (UK)	Female Shoe Size EU (UK)
79-81513	Diabetic Shoe, Small	36-38.5 (3.5-5.5)	37-39 (4-6)
79-81515	Diabetic Shoe, Medium	39-42 (6-8)	39.5-42.5 (6.5-8.5)
79-81517	Diabetic Shoe, Large	42.5-45 (8.5-10.5)	43-44.5 (9-10)
79-81518	Diabetic Shoe, XLarge	46-48.5 (11-13.5)	45+ (10.5+)

Part Number	Description
79-81523	Small IMPAX Insole
79-81525	Medium IMPAX Insole
79-81527	Large IMPAX Insole
79-81528	XLarge IMPAX Insole

*Pilot testing has shown the IMPAX insole enables effective off-loading of pressure from a specific area:

Without Diabetic IMPAX Insole

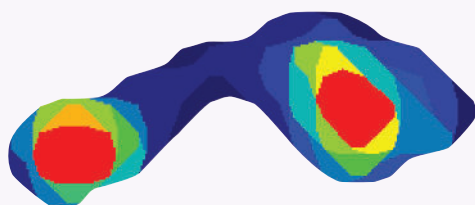


With Diabetic IMPAX Insole



Blocks were removed at site indicated

KPa

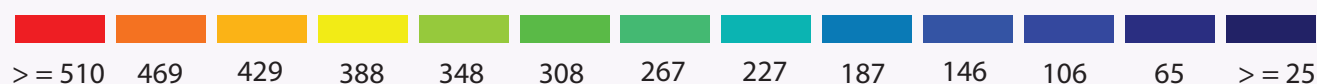

* Data on file

► APPLICATION OF DIABETIC SHOE AND INSOLE:



1. Remove insole from the shoe



2. The insole is made up of individual blocks



3. Individual blocks can be removed to enhance off-loading at a specific site



4. Top layer of the insole stays in contact with plantar surface to ensure no "hot spots"



5. Insole is replaced and shoe is ready for patient use



6. Shoe can be worn with or without the toe cover

ASIA-PACIFIC:

DJO Asia-Pacific Limited
Unit 1905, 19/F, Tower II
Grand Central Plaza
138 Shatin Rural Committee Road
Shatin
HONG KONG
Tel: +852 3105 2237
Fax: +852 3105 1444
Email: peter.hoyer@DJOglobal.com

BENELUX:

DJO Benelux BVBA
NCI Business Center
Twin Squares - Building Vendôme
Culliganlaan 1B
B-1831 Diegem
BELGIUM
Tel: +32 2 403 1320/19
Fax: +32 2 403 1212
Email: benelux.orders@DJOglobal.com

CANADA:

DJO Canada
2835 Argentinia Road, Unit # 5
Mississauga
ON L5N 8G6
CANADA
Tel: + 1 1866 866 5031
Fax: + 1 1866 866 5032
Email: miles.white@DJOglobal.com

**DENMARK, FINLAND,
NORWAY & SWEDEN:**

DJO Nordic AB
Murmansgatan 126
21225 Malmö
SWEDEN
Tel Sweden: 020 794 950
Tel Norway: 8006 1052
Tel Finland: 0800 114 582
Tel Denmark: +45 4541 0670
Email: info.nordic@DJOglobal.com

FRANCE:

DJO France S.A.S.
Centre Européen de Fret
Mouguerre
64990
FRANCE
Tel: +33 (0)5 59 52 86 90
Fax: +33 (0)5 59 42 86 91
Email: sce.cial@DJOglobal.com

GERMANY:

ORMED-DJO Germany GmbH
Merzhauser Str. 112
D-79100 Freiburg
GERMANY
Tel: +49 761 4566 251
Fax: +49 761 4566 55251
Email: www.ormed-djo.de

ITALY:

DJO Italia Srl
Via Leonardo Da Vinci 97
Trezzano Sul Naviglio
20090 Milano
ITALY
Tel: +39 02 484 63386
Fax: +39 02 484 09217
Email: it.info@DJOglobal.com

SPAIN & PORTUGAL:

DJO Ibérica
Carretera de Cornellá 144, 1º 4ª
Esplugues de Llobregat
08950 Barcelona
SPAIN
Tel: +34 943 638 167
Fax: +34 943 638 174
Email: svc.cial@DJOglobal.com

UK & IRELAND:

DJO UK Ltd.
7 The Pines Business Park
Broad Street
Guildford
Surrey
GU3 3BH
ENGLAND
Tel: +44 (0)1483 459 659
Fax: +44 (0)1483 459 470
Email: ukorders@DJOglobal.com

ALL OTHER INTERNATIONAL - DJO EXPORT:**UNITED STATES:**

DJO, LLC
1430 Decision Street
Vista
CA 92081-8553
U.S.A.
Tel: 1 800 336 6569
Fax: 1 800 936 6569

CHATTANOOGA GROUP

4717 Adams Road
Hixson
TN 37343
USA
Tel: +1 423 870 7200
Fax: +1 423 870 204
Email: juan_villegas@chattgroup.com

CHATTANOOGA EUROPE

Welvaartstraat 8
2200 Herentals
BELGIUM
Tel: +32 (0) 14248350
Fax: +32 (0) 14248358
Email: hans_ploemen@chattgroup.com



NEVER STOP GETTING BETTER®