



TRAUMA COLLAR QUICK CLIP PATIENT QUICK START GUIDE

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, or the patient's caretaker providing assistance. User should be able to: read, understand and be physically capable of following all the directions, warnings and cautions provided in the instruction for use.

QUICK CLIP VERSION

Ambulatory (upright) Application:

Step 1: Check the Quick Clip Straps

Check the quick clip straps on back panel to ensure the Sizing Letters match on each side. (Fig. A).

Step 2: Attach the Quick Clip on One Side

Attach the quick clip to one side of the corresponding clear plastic slot on the front panel until you hear a click (Fig. B)

Step 3: Place the Front Panel to the Chin

While looking in a mirror and maintaining chin level parallel to the floor, place the front panel of the collar to the chin, holding it in place with one hand at the chest (Fig. C)

Step 4: Position the Back Panel




While stabilizing/balancing the front of the collar with one hand, reach behind your neck with your other hand to grab the back panel of the collar and position it behind your neck. (Fig. D)

Step 5: Attach the Quick Clip

Attach the remaining quick clip to one side of the corresponding clear plastic slot until you hear a click (Fig. E)

Step 6: Check Fit


Ensure the collar quick clips are secured, the chin is positioned near the edge of the foam, and the height of the collar maintains a neutral neck position.

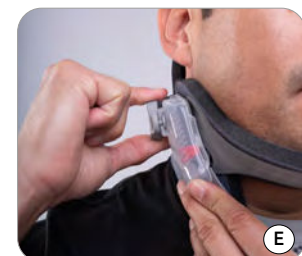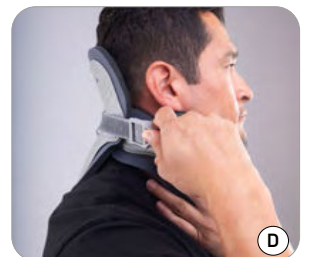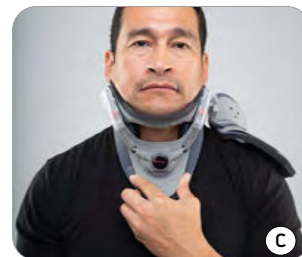

TRAUMA COLLAR QUICK CLIP PATIENT QUICK START GUIDE


Step 7: Adjust Height (if needed)

If chin height adjustment is needed, hold the front panel of the collar with one hand, while pulling the dial knob out. Turn the dial counter-clockwise to increase height, or clockwise to lower chin height. Then, push the dial inwards to lock it in place. Be sure to maintain your chin level parallel to the floor (Fig. F)

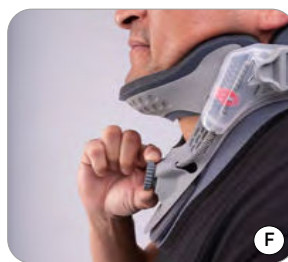

Step 8: Removal (when needed)

To remove the collar, hold the front of collar across the sternum with one hand while pulling one side of the quick clip tab forward and out to release it from the slot. Then move the collar away from your chin. (Fig. G)

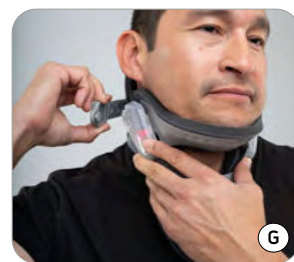

MATERIAL CONTENTS:

Nylon, Polyester, Propylene Ethylene Copolymer, Thermoplastic Polyester Elastomer, Polyurethane, Polyethylene.

WARRANTY: 1 year from time of purchase.

NOT MADE WITH NATURAL RUBBER LATEX

CLEANING INSTRUCTIONS

Hand wash in warm water (30° C/86°F) using mild soap and rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate materials.

RX ONLY

CONTRAINDICATIONS:

Do not use if you are allergic to any of the materials contained within this product.

WARNINGS AND CAUTIONS:

- This product is to be prescribed and fitted by a physician.
- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- The collar should be snug but not tight enough to cause excessive pressure, pain or difficulty breathing or swallowing.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately.
- There could be difficulties with laryngoscopy when collars are used; hence, the device may have to be removed.



APPLICATION
VIDEO

enovisTM

Creating Better Together™

enovis-medtech.eu

2000 Cathedral Square, Cathedral Hill, Guildford, Surrey, GU2 7YL, UNITED KINGDOM

Individual results may vary. Neither Enovis, DJO, LLC or any of their subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

QR Code on form is directed to an application video on the corporate YouTube channel.